
Montage handleiding SecuCare Wandbeugel 45 & 90 graden

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Assembly manual for SecuCare Wall Grab Rail, 45 & 90 degrees

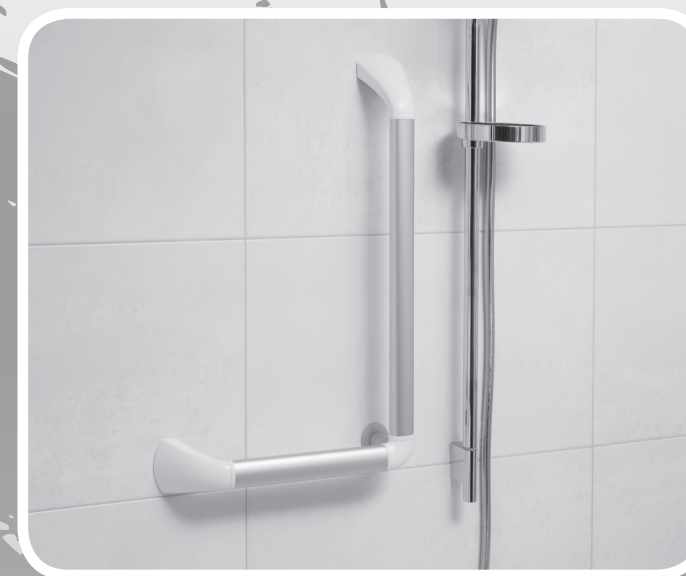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### Required tools :

Electric drill/screwdriver  
 Masonry drill, 8 and 10 mm  
 Allen bolt, 6 mm  
 Cap no. 10 (for driving electric drill or ratchet)  
 Pozidrive screwdriver  
 Optional hacksaw / lubricating grease

### Parts list, 45 degrees:

2 grip sections, aluminium 231 mm  
 2 mounting legs, grey  
 2 covers, white  
 2 sealing rings  
 1 coupling piece, grey 45 degrees  
 1 cover, white  
 1 aluminium foot, round  
 6 SS chipboard screws, 6x40  
 6 plugs, diam. 8 mm (for masonry)  
 1 SS Allen bolt, M8 x 60 mm  
 1 serrated lock washer, diam. 8 mm  
 1 Fischer MS Plug M8 (for masonry)  
 3 Taptite SS hexagonal (no. 10)  
 2 Taptite SS Pozidrive

### Parts list, 90 degrees left & right:

1 grip section, aluminium 231 mm  
 1 grip section, aluminium 431 mm  
 2 mounting legs, grey  
 2 covers, white  
 2 sealing rings  
 1 coupling piece, grey 90 degrees  
 1 cover, white  
 1 aluminium foot, round  
 6 SS chipboard screws, 6x40  
 6 plugs, diam. 8 mm (for masonry)  
 1 SS Allen bolt, M8 x 60 mm  
 1 serrated lock washer, diam. 8 mm  
 1 Fischer MS Plug M8 (for masonry)  
 3 Taptite SS hexagonal (no. 10)  
 2 Taptite SS Pozidrive

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## GB

**Note: the wall grab rail will not attain its full strength until mounted on the wall; therefore caution is advised during assembly.**

1. Check the contents of the packaging to see if everything is complete.
2. Place the white sealing rings over the mounting legs and attach them to the aluminium grip sections using the supplied Taptite screws (hexagonal version). Tighten them firmly! (fig. 1)

final result fig. 2

3. Place the preassembled grip sections in the 45 or 90 degree coupling piece. (fig. 3)

4. Now check the planned mounting location to see if any of the aluminium grip sections should be shortened. Attention: for the 90 degree model, check if you need a left-hand (fig. 4) or right-hand (fig. 5) version.

5. If the grip sections do not have to be shortened, continue to point 7.

6. Shortening the grip sections:  
Determine the desired length and shorten with a good hacksaw (fig. 6).

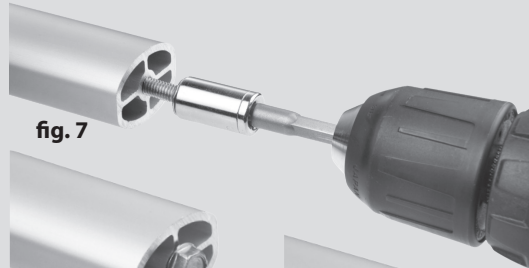
fig. 7

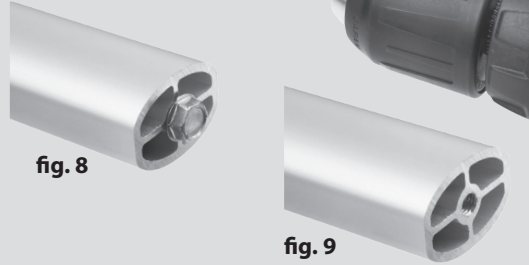
fig. 8

fig. 9

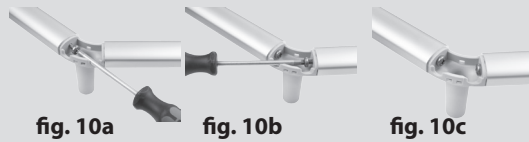
fig. 10a

fig. 10b

fig. 10c

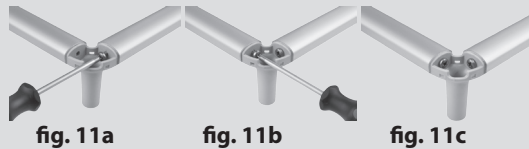
fig. 11a

fig. 11b

fig. 11c

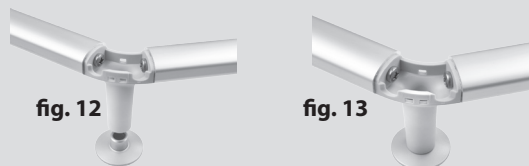
fig. 12

fig. 13

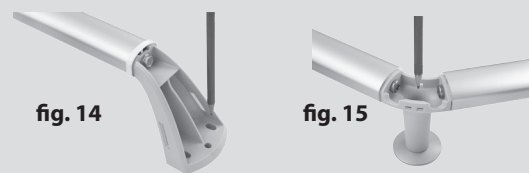
fig. 14

fig. 15

fig. 16a

fig. 16b

fig. 16c

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You must now first recreate the screw thread using the additionally supplied hexagonal Taptite screw. This screw is self-tapping and can be reused.

Lightly grease the screw and slowly screw it into the cylindrical hole present using the electric screwdriver (with cap no. 10). Firmly hold the grip section.

fig. 7: Screw in  
fig. 8: Screw fully inserted  
fig. 9: Screw removed, you can see the tapped screw thread

7. Now attach the grip sections permanently to the angled coupling piece using the Taptite Pozidrive screws. Tighten them firmly!

fig. 10a, 10b and 10c for 45 degrees.  
fig. 11a, 11b and 11c for 90 degrees.

8. Slide the aluminium foot into the bottom of the coupling piece.  
fig. 12 and fig. 13.

9. Now determine the definitive locations of the holes to the drilled and mark them.  
fig. 14 and fig. 15.

10. Pre-drill the mounting holes and insert the plugs (mounting legs plastic 8 mm, aluminium foot 10 mm).

11. Attach the wall grip (mounting legs plastic chipboard) using screw 6x40, the coupling piece using the Allen bolt (with serrated lock washer) M8 x 60 mm.  
Fig. 16a, 16b and 16c.

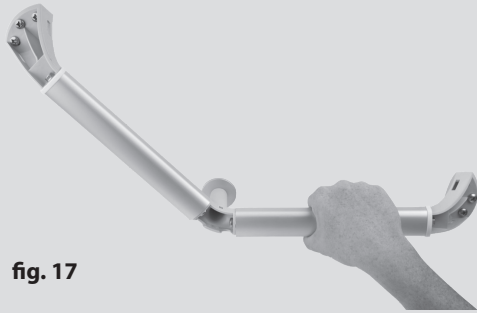
fig. 17

fig. 18a

fig. 18b

fig. 18c

fig. 19a

fig. 19b

fig. 20a

fig. 20b

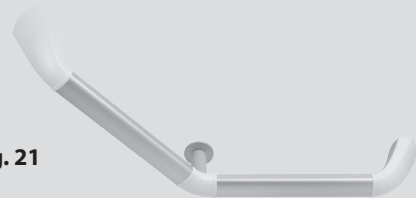
fig. 21

fig. 22

fig. 23

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fig. 23, droite 90 degrés

12. 12. Check the fastening (fig. 17),

13. if it is correct, click the covers in place.

Fig. 18a, 18b and 18c.

Fig. 19a and 19b (90 degrees)

Fig. 20a and 20b (45 degrees)

14. For your own safety, regularly check if the fastening is still OK.

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#### Disclaimer:

#### Cleaning advice

Clean daily with a soft towel and, if necessary, with a non-aggressive, non-scouring detergent (e.g. green soap). Do not clean using bleach and/or descaling agents. After cleaning, rinse well and dry. Ensure no traces of detergent remain.

#### Guarantee

Please keep your proof of purchase with the date of purchase and product description or name stated on it: this is your guarantee certificate. The five-year guarantee commences after the date of purchase by the end-user. The guarantee only applies in the event of production errors in the materials used in the SecuCare product. A successful claim under the guarantee will result to no entitlement other than delivery of a replacement product by SecuProducts BV free of charge. SecuProducts BV accepts no liability whatsoever for any injury or consequential damage that arises during the mounting and/or use of the product.

#### The guarantee will lapse through:

- excessive pressure placed on the product or the incorrect, inappropriate or unnatural use of the product;
- damage caused deliberately or through neglect;
- application on an uneven surface;
- damage caused by lack of maintenance, incorrect maintenance or the use of unsuitable cleaning agents (see cleaning advice);
- damage caused during mounting;
- incorrect mounting (including mounting that is not in accordance with the mounting instructions or use of mounting material other than that supplied);
- influence of chemical/electro-chemical substances or electricity;
- damage caused by 'force majeure' or natural disasters (including flooding, fire and freezing).

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